# LATITUDE™ Communicator Patient Manual Supplement

## Hotspot Feature and Wireless Internet Adapter

EN

### Wireless Internet Adapter


This supplement to your patient manual explains how to connect your LATITUDE Communicator to the Internet (using the LATITUDE Wireless Internet Adapter). The steps below assume that your Communicator is already set up and connected to an electrical outlet.

**Prerequisites:** To use the wireless internet adapter, you must have the USB Accessory Adapter plugged into your Communicator. In addition, position the Communicator within 100 Feet (30 meters) of the wireless internet adapter.

1. Insert the micro end of the USB power cable, included with the wireless internet adapter, into the port on the back labeled "USB."
2. Insert the standard (narrow) end of the USB power cable into the USB power adapter, included with the wireless internet adapter.
3. Plug the USB power adapter into an electrical outlet.
4. Insert the Ethernet cable, included with the wireless internet adapter, into the port on the back of the wireless internet adapter labeled "ETHERNET."
5. Plug the Ethernet cable into an Ethernet port for your Internet service, such as a modem, router, or Ethernet wall jack.
6. To begin pairing, press the button on the back of the wireless internet adapter. The  indicator on front will flash green to signal pairing mode is active.
7. When the Heart button flashes on the front of your Communicator, press it.

**Note:** If you have previously completed initial setup, the Heart button will not flash at this point. Instead, press and hold the Status button on the back of the Communicator for at least 3 seconds to complete pairing.

8. Your Communicator has successfully connected to the LATITUDE system if the wave lights on your Communicator are lit a solid green.
9. Refer to your patient manual for additional instructions.
10. To return or dispose of the wireless internet adapter, follow the instructions in your patient manual.

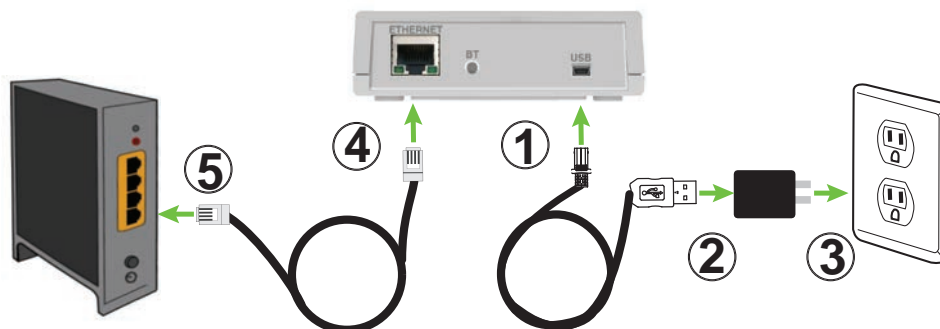---

# LATITUDE™ Communicator Patient Manual Supplement

## Hotspot Feature and Wireless Internet Adapter

---

EN

### Hotspot Feature

This supplement to your patient manual explains that you can use your mobile device as a wireless connection method for your LATITUDE Communicator. The instructions below assume that your Communicator is already set up and connected to an electrical outlet.

To use hotspot, you must have the USB Accessory Adapter plugged into your Communicator. The pairing PIN number is "123456".

Enable hotspot and *Bluetooth*® on your mobile device while near your Communicator for at least 1 hour each day and for manual device interrogations. This allows enough time for the Communicator to contact the LATITUDE system. You may incur charges from your mobile phone service provider. For additional assistance, contact your mobile phone service provider or LATITUDE Patient Services at 1-866-484-3268.

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